MASTER-AID®

IT

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EN

PREVENTION AND TREATMENT OF HYPERTROPHIC SCARS

SKIN REPAIR

Following a wound of any sort, a healing process begins which concludes with scar formation. The healing phase takes anywhere from 21 days to two years.

HYPERTROPHIC SCARS

Hypertrophic (and keloid) scars are the result of excessive collagen production invading the intact skin around the injured area. Scar formation takes place during the healing phase.

INDICATIONS

Resolve Cerotto Siliconico Patches allow for correct management of the healing phase. **Resolve Cerotto Siliconico** aims to prevent the formation of hypertrophic scars. **Resolve Cerotto Siliconico** can be used on the scars produced by any type of wound (surgical, caesarean section, abrasions, scrapes, burns, animal bites, laceration, cuts). It is recommended that treatment continues for at least 2 months for the best aesthetic result.

WHAT IS RESOLVE CEROTTO SILICONICO

Resolve Cerotto Siliconico is a self-adhesive silicone bandage (medical device) consisting of a skin-colored support tape. Its semi-occlusive and hydrating action keeps the healing environment at a constant temperature and prevents moisture loss from the tissue.

HOW TO USE

Resolve Cerotto Siliconico can be applied and left on the skin for the entire day. For best results use for at least 12 hours and up to 23 hours a day. It can be removed while the treated area is being washed and later reapplied. **Resolve Cerotto Siliconico** can be reused many times. The number of uses varies from person to person and depends on the area on which it is applied. Replace **Resolve Cerotto Siliconico** when it begins to lose its adhesive capacity and shows wear. When used e conserved properly, it should last for around three weeks.

INSTRUCTIONS FOR USE

1. Clean and dry the scar and the surrounding skin carefully.
2. Open the pouch of **Resolve Cerotto Siliconico** and store it in the pack.
3. Remove the transparent film from the cohesive side and store it in the pouch.
4. Apply **Resolve Cerotto Siliconico** to the scar.
5. In case of heavy sweating and / or extended use please fix **Resolve Cerotto Siliconico** with an adhesive bandage or another fixing device (cohesive bandage, elastic net bandage).
6. **Resolve Cerotto Siliconico** can be removed and reused later: in this instance, place the adhesive side of **Resolve Cerotto Siliconico** on the transparent film and put it back into the pouch.
7. Reposition **Resolve Cerotto Siliconico** onto the scar, repeating steps 1–2–3.
8. **Resolve Cerotto Siliconico** can be washed using warm running water and mild soap (this improves the adhesiveness and hygiene of the product).
9. To dry **Resolve Cerotto Siliconico**, lay it on a clean dry cloth with

CEROTTO SILICONICO

- the adhesive side facing up.
10. Only reuse **Resolve Cerotto Siliconico** when it is completely dry.
11. Wash the patch daily.

⚠️ WARNINGS

Resolve Cerotto Siliconico must be applied only on wounds that have closed and are in the process of healing. In the case of maceration, reddening or rash, discontinue treatment until the symptoms have disappeared. **Resolve Cerotto Siliconico** is contraindicated in subjects with dermatological disorders incompatible with long term use of the product. When a patch has already been applied to one user, it must not be applied to a second user.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Reference Member State.

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WAS IST RESOLVE CEROTTO SILICONICO

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